

Steps to lancing success

1 Prepare



Twist and unlock the endcap as shown
Line up the white raised mark and black indented mark



Loosen the round protective cap on the lancet but do not remove it
Rotate 1/4 turn



Insert the lancet firmly into the lancing device
Push until the lancet comes to a complete stop



Carefully twist off the round protective cap
Save the cap for disposing of the used lancet



Re-attach the endcap
'Click' to lock

2 Lance



Rotate the endcap dial to adjust the puncture depth
1 for the shallowest and 5 for the deepest



Pull back the reset handle until the release button is raised



Press the lancing device firmly against the side of your finger and press the release button
You will hear a click when the lancet is released

3 Eject



Remove the endcap
Place the protective cap that you saved onto a flat surface. Push the lancet into the cap as shown



Push the grey lancet ejector forward until the lancet falls into the container, then slide ejector back
Don't forget to re-attach the endcap



Before use and for full information please see the MICROLET®NEXT packaging insert



Microlet
next Lancing Device

ASCENSIA
Diabetes Care

Ascensia Diabetes Care Switzerland AG
Peter Merian-Strasse 90
4052 Basel

Tel.: 061 544 79 90
E-Mail: info@ascensia.ch
www.ascensia-diabetes.ch

Contour
diabetes solutions

ASCENSIA
Diabetes Care

**CONTOUR®XT meter and
CONTOUR®NEXT test strips**

Quick reference guide

Simple. Precise.

multi
puls

NO CODING



Contour
next
Blood Glucose Test Strips

Contour
XT
Blood Glucose
Monitoring System

STEP 1: Insert test strip

Insert the grey end of the test strip into the test strip port of the CONTOUR[®]XT blood glucose meter. This automatically codes the meter. The meter will switch on and a test strip with a flashing drop of blood will appear in the display. This indicates that the device is ready to measure blood glucose.



STEP 2: Test blood + display test result

Immediately touch the opening of the test strip to the drop of blood. The blood is automatically pulled into the test strip. Hold the test strip in the drop of blood until the device beeps.

The beep indicates the start of the five-second test time. Your test result is then displayed and automatically saved.



STEP 3: Display test result

The beep indicates the start of the five-second test time. Your test result is then displayed and automatically saved.



STEP 4: Remove test strip

The device switches off automatically as soon as you remove the used test strip from the CONTOUR[®]XT.



CONTOUR[®] NEXT test strips

CONTOUR[®]NEXT test strips are available in packs of 50 or 100.

